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Standby HOST 37.0 2017-09-05 8:20:55 AM ? V

Reagent Calibration QC Status Statistics Parameters Setup Maintenance

Test Profile Calculation Off-system Carryover

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Calibrators and concentration for selected test

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SD=(Max - Min) / 4

Controls and concentration for selected test

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Relog

Exit

Test → Profile
Calculation
Off-system
Carryover

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Select tests to included in selected profile

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Test Profile **Calculation** Off-system Carryover

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The screenshot shows the BS-200E Mindray software interface. The top bar includes the logo, version (Demo Version), host name, and date/time. The main menu has icons for Reagent, Calibration, QC, Status, Statistics, Parameters (selected), Setup, and Maintenance. The left sidebar contains buttons for Sample Request, QC Request, Start, Probe Stop, Sleep, Results, Replace, Relog, and Exit. The main window is titled 'Test' and shows a list of tests on the left: ALL, Glu_R1, urea_R1, urea_R2, Creat_R1, and Creat_R2. The right pane displays the 'Carryover' settings for the selected test. It includes a table with columns for ALL, Glu_R1, urea_R1, urea_R2, Creat_R1, and Creat_R2. Below the table, there is a section for 'Select a test that you want to view or change the carryover relationship' with buttons for 'ثبت تنظیمات' (Save Settings) and 'لغو تنظیمات' (Cancel Settings). The bottom bar has an 'Admin' button and navigation controls.

Test | Profile | Calculation | Off-system → Carryover

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Select a test that you want to view or change the carryover relationship

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R2
Position: . .
Bottle Type: .
Estimated:
Exp. Date: 2018-09-04
Lot No.:
Bottle No.:
Bar Code:

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[Demo Version] HOST 37.0 2017-09-05 7:58:58 AM ? V

Reagent Calibration **QC** Status Statistics Parameters Setup Maintenance

Real-time Daily QC Day to Day QC QC Sum. **Control**

Controls

Name	Lot No.	Position	Exp. Date	Level
Control N	Lot No.	1- 38	2018-08-21	Medium
Control H	Lot No.	1- 37	2018-09-05	High

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Standby HOST 37.0 2017-09-05 7:59:56 AM ? V

Reagent Calibration QC Status Statistics Parameters Setup Maintenance

Sample Request
QC Request
Start
Probe Stop
Stop
Results
Replace
Relog
Exit

Real-time Daily QC Day to Day QC QC Sum. Control

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The screenshot shows the BS-200 mindray software interface. The top bar includes 'Standby', 'HOST', '37.0', and the date/time '2017-09-05 8:00:32 AM'. The main menu has icons for Reagent, Calibration, QC (highlighted), Status, Statistics, Parameters, Setup, and Maintenance. The left sidebar contains 'Sample Request', 'QC Request', and buttons for 'Blank', 'Print', 'Stop', 'Results', 'Replace', 'Reclog', and 'Exit'. The main window has tabs for 'Real time', 'Daily QC', 'Day to Day QC', 'QC Sum.', and 'Control'. The 'Daily QC' tab is selected, showing a 'Test' field, a 'Date' dropdown set to '2017-09-05', a 'Rule' dropdown set to 'Westgard Multi-rule', and two 'Control' dropdowns set to '1' and '2'. Below these is a table with columns: Test, Control, Mean Conc., SD, Unit, Actual Result, and Request Date. At the bottom, there are buttons for 'Refresh', 'چاپ نتایج', 'حذف نتایج روز جاری', 'مشاهده نمودار واکشی', and 'ارسال نتایج به LIS'.

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The screenshot shows a 'Confirm' dialog box. The text inside reads: 'It will take a long time to search the QC data and the screen probably can't be refreshed during this period. Do you want to continue?'. There are 'OK' and 'Cancel' buttons at the bottom.

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The screenshot shows the BS-200 mindray software interface. The top bar includes 'Standby', 'HOST', '37.0', and the date/time '2017-09-05 8:01:47 AM'. Below this is a navigation bar with icons for Reagent, Calibration, QC (selected), Status, Statistics, Parameters, Setup, and Maintenance. The main window has a left sidebar with buttons: Sample Request, QC Request, Start, Pause/Stop, Stop, Results, Replace, Rolog, and Exit. The main area is divided into tabs: Real-time, Daily QC, Day to Day QC (selected), QC Sum., and Control. Under 'Day to Day QC', there are options for 'Tabular Data' (selected) and 'Graphical Data'. Below these are dropdowns for 'Test' (selected), 'Date' (2017-09-05), 'Rule' (Westgard Multi rule), 'Control 1' (1 انتخاب کنترل), and 'Control 2' (2 انتخاب کنترل). A table with columns 'Test', 'Control', 'Mean Conc.', 'SD', 'Unit', 'Actual Result', and 'Request Date' is shown, but it is empty. At the bottom, there are buttons for 'Refresh', 'Print', 'Delete', 'Export', and 'Import'. The bottom status bar shows 'Admin' and navigation arrows.

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A 'Confirm' dialog box with the text: 'It will take a long time to search the QC data and the screen probably can't be refreshed during this period. Do you want to continue?'. At the bottom are 'OK' and 'Cancel' buttons.

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The screenshot shows the BS-200 mindray software interface. The top menu bar includes options like Reagent, Calibration, QC, Status, Statistics, Parameters, Setup, and Maintenance. The main area displays 'QC Sum' with tabs for Real-time, Daily QC, Day to Day QC, and QC Sum. Below these are fields for Date (2017-09-05), Control (انتخاب کنترل), and Test Statistics. A table with columns Test, Actual Result, Unit, Request Date, Ref. Range, and Comments is shown. At the bottom, there are buttons for Refresh, Print, Delete, and a 'Clear' button in the Admin section.

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The screenshot shows a 'Confirm' dialog box. The text inside reads: "It will take a long time to search the QC data and the screen probably can't be refreshed during this period. Do you want to continue?". There are 'OK' and 'Cancel' buttons at the bottom.

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The screenshot shows the BS-200E Mindray software interface. The top menu bar includes options like Resagent, Calibration, QC, Status, Statistics, Parameters, Setup, and Maintenance. The main area is divided into several sections: Sample Request (with a 'Sample Disk' dropdown), Sample Information (with fields for No., Position, Patient, and Type), Tests (a table for selecting tests like GLU, Creat, Urea), and a 'جدول انتخاب تست' (Test Selection Table). There are also buttons for 'Change Pos.', 'Print List', and 'Exit'. A red arrow points to the 'Sample Request' button on the left sidebar. Another red arrow points to the 'STAT' checkbox in the Sample Information section. A third red arrow points to the 'Change Pos.' button at the bottom. The interface is in Persian and includes various labels and instructions in both Persian and English.

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Standby HOST 37.0 2017-09-10 8:33:23 AM ? V

Reagent Calibration QC Status Statistics Parameters Setup Maintenance

Sample Request
QC Request
Start
Probe Stop
Stop
Results
Replace
Releg
Exit

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Search Add Details Reac.Curve Delete Rerun Print

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BS-200E
mindray

Standby. Requested tests exist HOST 37.0 2017-09-04 2:59:44 PM ? V

Reagent Calibration QC **Status** Statistics Parameters Setup Maintenance

Sample Disk Reagent Disk Reaction Disk

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Details Refresh Change Pos. Release All

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BS-200E mindray

Standby. Requested tests exist HOST 37.0 2017-09-04 4:17:05 PM ? V

Reagent Calibration QC **Status** Statistics Parameters Setup Maintenance

Sample Request QC Request Start Probe Stop Stop Results Replace Relog Exit

Sample Disk → Reagent Disk Reaction Disk

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Refresh All Refresh Change Pos. Inventory Release All Release Pin

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The screenshot displays the BS-200 Mindray software interface. At the top, there's a status bar with 'Testing (13s)', 'HOST', '37.0', and the date/time '2017-09-05 8:12:25 AM'. Below this is a navigation menu with icons for Reagent, Calibration, QC, Status (highlighted), Statistics, Parameters, Setup, and Maintenance.

The main area is divided into three sections: Sample Disk, Reagent Disk, and Reaction Disk (selected). The Reaction Disk shows a circular layout of 80 wells, numbered 1 to 80. A legend in the center lists various test statuses in Persian, each with a corresponding color-coded box. To the right of the disk, there's a table with test details:

Position	1	Status	Finished
Type	Calibration		
Test	urea		
Calibrator Information			
Name	Calibrator1		
Conc.	40		
Lot No.	Lot No.		
Position	1	39	
Level	Medium		
Exp. Date	2018-09-04		
Test Data			
Start	2017-09-05 8:11:45 AM		
End	2017-09-05 8:12:22 AM		

At the bottom, there are buttons for 'Rerun' and 'Reac. Curve', and an 'Admin' section with a 'Clear' button.

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The screenshot displays the 'Setup' window of the BS-200 mindray software. The interface is in Persian. The top bar shows 'Standby', 'HOST', '37.0', and the date '2017-09-05 8:28:18 AM'. The main menu includes 'Reagent', 'Calibration', 'QC', 'Status', 'Statistics', 'Parameters', 'Setup' (highlighted), and 'Maintenance'. The 'Setup' window has tabs for 'System', 'Hospital', 'User', and 'Print'. The 'Basic' tab is active, showing various settings like 'Number of Rgt. Disks', 'Wash When Started Up', 'Reagent Alarm Limit', 'Dilution Ratio', 'Sample Volume', and 'QC Prompt'. There are also checkboxes for 'Auto Rerun Setup' and 'Wash' settings. The bottom of the window has 'Restore', 'OK', and 'Cancel' buttons.

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Standby HOST 37.0 2017-09-05 8:30:20 AM ? V

Reagent Calibration QC Status Statistics Parameters Setup Maintenance

Sample Request QC Request Start Probe Stop Stop Results Replace Relog Exit

Daily Maint Log Import/Export Alignment

System Status

Parameter	Value
Reaction Temp.	37.00
Preheat Temp.	
Waste Tank	
Deionized Water	
Printer	Idle
Light Intensity	68%
Unit Status	
Main Unit	
Reaction Unit	
Sample Unit	
Reagent Unit	
Temperature Unit	
Mixing Unit	

Dark Current/Background

Dark Current	Background
340	44765
405	44765
450	44765
510	44765
546	44765
578	44765
630	44765
670	44765
Reference	44765

Startup Check Wash Enhanced Dark Current Reset

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